ONE

onecondoms.com

Manufacturer, Fabricante, Fabricant, Hersteller, Produttore, Fabrikant, Tilverkärer, Kaxex Industries Sdn Bhd PTD 7906 & 7907, Taman Pontian Jaya, Batu 34, Jalan Johor, 82000 Pontian, Johor, Malaysia

Brand Owner, Proprietario de la marca, Propriétaire de la marque, Markeninhaber, Proprietario del marchio, Merkeigenaar, Varumärkesägare:

ONE®
630 Lynnway, Lynn, MA, 01905, USA
www.onecondoms.com

Authorized Representative, Representante Autorizado, Representant Autorisé, Bevollmächtigter Vertreter, Rappresentante Autorizzato, Gemachtigde Verteenwoordiger, Auktoriserat Ombud: Advena Ltd. at Tower Business Centre, 2nd Flr., Tower Street, Swatara, BKR 4013 Malta.

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0123

EN

Latex condoms are intended to prevent pregnancy, HIV/AIDS, and other sexually transmitted infections (STIs). Latex condoms reduce the risk of transmitting STIs by providing a barrier against the source of the infection. Condoms are most effective against STIs such as HIV infection (AIDS) and gonorrhoea that are spread by contact with the head of the penis. Condoms are less effective against STIs such as Human Papillomavirus (HPV) and Herpes. These STIs can also be spread by contact with infected skin that is not covered by the condom. If you believe you have an STI, contact a health care provider. For more information on latex condoms or STIs, contact a health care provider or public health agency. Always use a medical device intended to be used by lay persons.

⚠️ PRECAUTIONS:

Lesions, pre-ejaculate secretions, semen, vaginal secretions and blood can all transmit infectious organisms. Be sure to put on a condom prior to any sexual contact. Do not reuse condoms. Store condoms in a cool, dry place and avoid exposure to direct sunlight. If the latex material is sticky, brittle or obviously damaged, do not use the condom. If the color of the latex is not uniform or changed, do not use the condom. Make sure there is adequate lubrication. If you add lubricant, use a water-based or silicone lubricant. DO NOT USE OIL-BASED LUBRICANTS, such as those made with petroleum, mineral oil, vegetable oil, or cold cream, as these may damage the condom. **CAUTION: This product contains natural rubber latex, which may cause allergic reactions including anaphylactic shock.** If you or your partner are allergic to latex, consult a physician before use. Consult a doctor or pharmacist about the compatibility of topical medicines that may come in contact with the condom. Choking hazard: be careful if using condoms during oral sex. Keep away from children.

Report any serious incident occurred from using the device to the manufacturer and the relevant national competent authority.

© 2025, ONE® All Rights Reserved.
Tested according to EN ISO 4074.

PLEASURE PLUS®

For optimal pleasure, place the pouch at the tip on the underside of the penis.

GLOWING PLEASURES™

In order to glow, hold up to light for 30 seconds before use.

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**EN****1. HANDLE WITH CARE.**

1. Open up the package carefully. Do not use fingernails, teeth, or anything that can damage the condom. Use a new condom for every new act of intercourse. Never reuse condoms; re-use of condoms can result in condom breakage, risk of pregnancy and STIs.

2. PREPARE.

2. Remove the condom from the package and apply a small amount of lubricant to the inside tip. Using additional lubricant will reduce the risk of your condom breaking during use.

3. PINCH THE TIP

3. Before any sexual contact, place the condom on the head of the erect penis with the rolled side up. If the penis is not circumcised, pull the foreskin back before putting the condom on. Pinch the receptacle tip of the condom between your thumb and forefinger. This prevents air from becoming trapped at the tip of the condom and leaves an empty space to collect semen.

4. UNROLL.

4. Unroll the condom down the base of the penis with your other hand. If the condom doesn't unroll easily, it may be on backwards, damaged or too old. Throw it away and start over with a new condom.

5. APPLY LUBRICANT.

5. Using additional lubricant will reduce the risk of your condom breaking during use. Apply to the outside of the condom.

6. INTERCOURSE.

6. If the condom breaks during use, stop immediately and pull out. Do not continue until you put on a new condom and apply more lubricant. Stop and check if you feel the condom slipping off or tightening excessively onto the penis because this might lead to breakage. Seek medical assistance as soon as possible, at least within 72 hours, should a condom leak or burst during use.

7. HOLD ON TIGHT.

7. Immediately after ejaculation, hold on to the base of the condom tightly and pull out while the penis is still erect. This will keep the condom from slipping off and keep any fluids from being spilled.

8. DISPOSE PROPERLY. Wrap the used condom in tissue and throw it in the trash so others won't handle it. DO NOT FLUSH CONDOMS DOWN THE TOILET. Wash hands with soap and water. For added protection, wash genitals with soap and water.

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