FI - FINNISH

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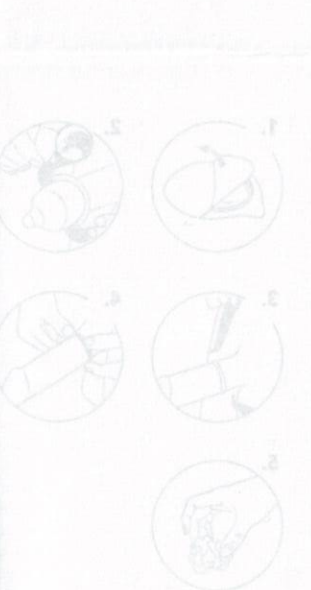
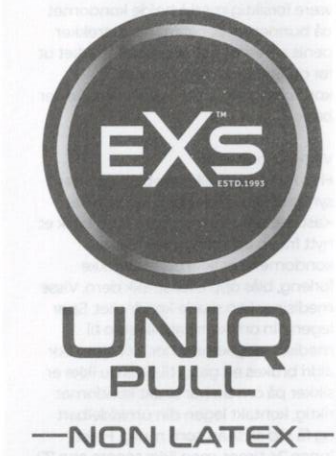
Made in Colombia by:
UNIQUE INTERNATIONAL
S.A.S, Cra 19A N° 196 - 23
Bogotá - Colombia

Authorized Representative in Europe:
EC REP MDI EUROPA GMBH
Langenhagener Str. 71
30855 Hannover-Langenhagen
Germany

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EN - ENGLISH

Application
Check that the condom packaging is intact and that it is not past its expiration date. Carefully open the package and remove the condom. Be careful not to damage it with your fingernails or jewellery. Take out the condom immediately before use.
Pull the condom over an erect and dry penis and pull the hoops towards the abdomen (do this before contact with the partner, as seminal fluid can come out even at the beginning of an erection). Do not use the condom if

you feel that it is slipping, contracting or pressing excessively on the penis, because this can lead to tearing.
Discard the hoops and use the condom. The condom will adapt to the penis. Make sure that the condom stays in position during sex. If the condom comes off, open a new condom and put it on before continuing with sex. If this happens, seek medical advice.
After ejaculation, you have to be careful to hold the condom on the base of the penis while you pull the penis out. Wait until the penis is completely pulled out before removing the condom.
Dispose of the condom hygienically in general waste after use.

Important information
If the condom or packaging has visible signs of damage, do not use it. Discard the damaged condom and use a new one from an undamaged package. These condoms are not elastic! Please do not extend, inflate or stretch them. Certain medications can damage the condom. Please ask your doctor about the compatibility of medications and condoms.
A condom should never be used a second time.
If you are not sure whether you have used the condom correctly, contact your doctor immediately and get advice as soon as possible. Preferably within 24 hours, but not later than 72 hours.

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