Use of Condom

Flip the tip of the capsule and carefully tear away the protection foil. The condom is already placed with the reservoir pointing upwards in the hygienic and safe capsule packaging. To protect from sexually transmitted diseases and unwanted pregnancy, the condom must be worn before any contact of penis and body openings. Hold the condom with thumb and forefinger on top of the reservoir, so that no air is inside and roll the condom completely over the erect penis. Be careful not to damage the condom with sharp objects such as jewellery, fingernails or teeth. After ejaculation, press and hold the condom at the base and withdraw still erect penis. Use each condom only once.
Warning: condom can slip, if penis is not erect

Use of Lubricants






When using additional lubricants, do not to use oil-based lubricants such as Vaseline, baby oil, body lotion, massage oil, butter, margarine etc., as these may damage the condom. Use only lubricants which are specifically recommended for use with condoms.

Storage

Store cool and dry. Avoid direct sunlight. Do not use condoms after expire date or if packaging is damaged.

Important:

When used correctly, these condoms can offer secure protection. No condom can offer 100% protection from sexually transmitted diseases and unwanted pregnancy. These condoms should not to be used by people with latex allergy. Medication, which is used in genital areas (e.g. ointments or creams) may harm the effectiveness of the condom. Please contact your doctor or pharmacist. Dispose used condoms hygienically, not in the toilette.
Warning: regrowing stubbles, extensive hair or genital piercings may damage the condom.

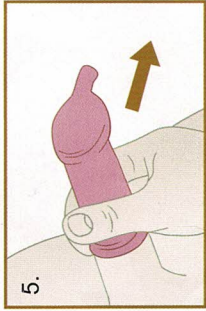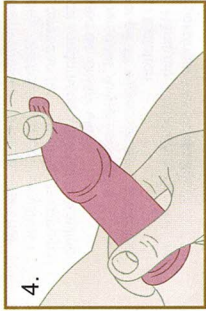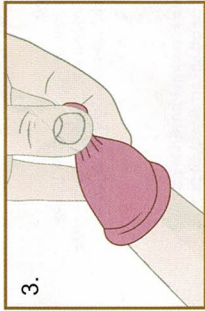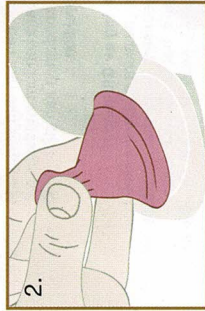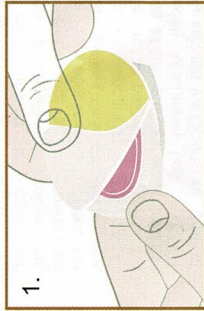Width: 52 ± 2 mm Length: ≥ 180 mm

Goldcaps AG
Seestrasse 345
8810 Horgen / Zurich
Switzerland

Authorized representative:
Shanghai International Holding
Corp. GmbH (Europe)
Address: Effestrasse 80,
20537 Hamburg, Germany

Manufacturer:
Ftione Latex Products Co., Ltd. Guangdong
Addr: No.5 Tongji Road, Lingbei Industrial Zone,
Suixi, 524338 Zhanjiang, Guangdong, China
Tel: (86) 759 7905898 Fax: (86) 759 7905869
Email: export@ftionelatex.com

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