## BRUGSANVISNING

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# FCUK

## INSTRUCTIONS FOR CONDOM USE

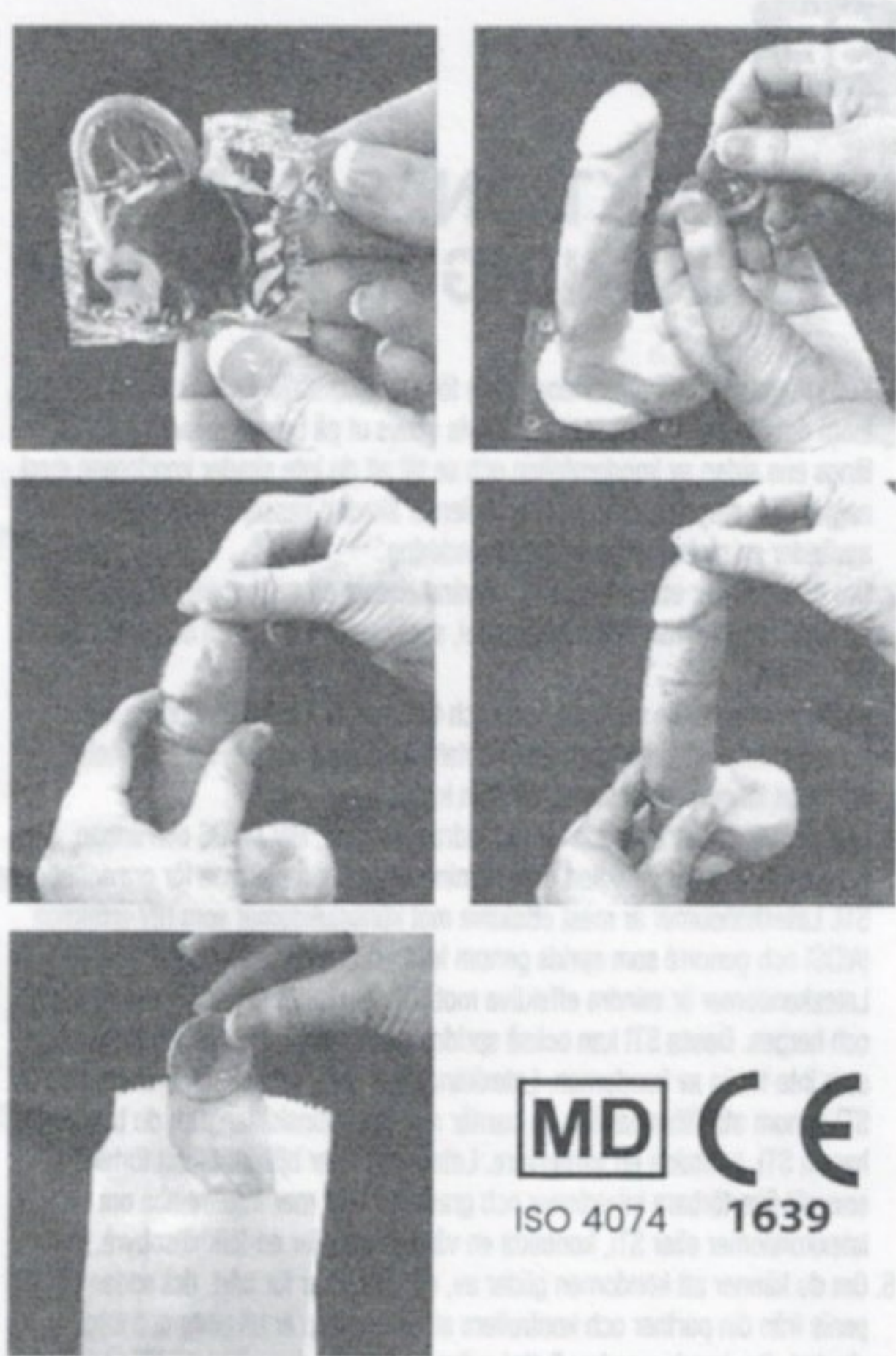
**FCUK**  
DISTRIBUTED BY:  
PROTECTION PLUS, PO BOX 729, MACCLESFIELD  
SK10 9DR

Socials - protectionplusgroup  
Email - info@protectionplusgroup.com  
www.protectionplusgroup.com

Customer Care Number Tel - 07904 017 408

LTC Healthcare Europe Limited  
Unit 3D, North Point House, North Point Business Park  
New Mallow Road  
Cork, T23 AT2P  
Ireland

EC REP rrt Vertrieb und Service GmbH,  
Heinrich Kümmler Str.3, 30169 Hannover, Germany.  
Richter Rubber Technology Sdn. Bhd.,  
Plot 209, Kuala Ketil Industrial Estate, 09300 Kuala Ketil, Kedah, Malaysia.

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# UK

## INSTRUCTIONS FOR CONDOM USE

- Check the date on the condom foil to ensure the condom is within the expiry date usually printed on the back of the foil. Tear along one side of the condom foil making sure you do not damage the condom with finger nails or jewellery. If the condom foil is damaged discard that condom and use a new one from an undamaged pack.
- If you are using a lubricant, only use a water-based lubricant. Do not use oil-based lubricants, such as petroleum jelly, as these may damage the condom.
- Hold the condom teat between your finger and thumb and roll the condom onto an erect penis before contact with your partner. Unroll down the full length of the penis expelling any air from the condom.
- Latex condoms are intended to prevent pregnancy, HIV/AIDS, and other sexually transmitted infection but do not completely eliminate the risks of pregnancy and STIs. Latex condoms are most effective against STIs such as HIV infection (AIDS) and gonorrhoea that are spread by contact with the head of the penis. Latex condoms are less effective against STIs such as Human Papillomavirus (HPV) and herpes. These STIs can also be spread by contact with infected skin that is not covered by the condom. Latex condoms reduce the risk of transmitting STIs by providing a barrier against the source of the infection. If you believe you have an STI, contact a health care provider. Latex condoms are to assist in prevention of sexually transmitted infections and pregnancy. For more information on latex condoms or STIs, contact a health care provider or a public health agency.
- If you feel the condom is slipping off, or is tightening excessively then withdraw the penis from your partner and check the condom is in place and not damaged. If the condom has moved then roll it back in place or use another condom if you are concerned about the condom being damaged. If the rubber material is sticky or brittle or obviously damaged, do not use the condom. If the colour is uneven or changed, do not use the condom. Condoms are made of natural rubber latex which may cause allergic reactions including anaphylactic shock if the user allergic to latex.
- After ejaculation withdraw the penis from your partner holding the base of the condom to ensure it does not slip off. When the penis is withdrawn from your partner you can then remove the condom.
- If you are concerned the condom has been damaged during sex then seek medical assistance such as a Doctor or Sexual Healthcare advisor preferably within 24 hours, but not later than 72 hours. If the condom foil is damaged discard that condom and choose another condom from the pack. Consult a doctor or pharmacist about the compatibility of topical medicines that might come in contact with the condom.
- Use a condom only once. Re-use of a condom may increase the risk of breakage, slippage, transmission of sexually transmitted infections including HIV. Dispose of the condom by wrapping it in tissue and placing in a waste bin. Store any unused condoms in a cool dry place away from direct sunlight.

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