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(EN) The Swiss brand you trust
ceylor stands for responsible, sensual and varied love lives. For 100 years ceylor has been the trusted Swiss brand. ceylor strives to offer the right product of the highest quality for every need and for every time of life.

Play it safe
ceylor latex condoms comply with the European standard EN ISO 4074. Furthermore, all ceylor condoms have been awarded the Swiss OK seal from the «Quality Seal for Condoms» association. This means that they are required to fulfil the world's strictest quality requirements, far in excess of the normal legal standards.

Safer sex rules
Stay safe and always stick to these safer sex rules:

- Sex with a condom
- And because everybody loves different things: do the personalised safer sex check at lovelife.ch

Unique – the condom pod.
Experience greater safety thanks to the exclusive condom pod for a carefree love life. Each individual condom enjoys perfect all-round protection and is held in place ready for rolling on. This means: No tears, no creases, no complicated opening, no fiddly unrolling.

Correct application
Sex is fun, also with a condom. How it works:

1. Be careful to avoid damaging the condom with sharp nails, rough skin, jewellery, etc.
2. Only open the condom pod immediately before use. Bend the foil tip on the pod, so that you can carefully remove the foil and then gently take out the condom between your thumb and forefinger. Only draw the condom over the penis once it is fully erect, but before it comes into contact with your partner's mucosa for the first time.
3. Make sure that the condom is unrolled the right way round. Place the condom ring on the penis such that the rolled rim is on the outside. If you have attempted to unroll the condom the wrong way round, use a new one. Also ensure that no seminal fluid comes into contact with the condom's exterior.
4. Hold the condom tip between the thumb and forefinger. Lightly press the thumb and finger together so that no air remains in the tip.
5. Now draw the foreskin back slightly and roll the condom down the entire length of the erect penis using the other hand.
6. Carefully withdraw the penis shortly after ejaculation and before sleeping. When doing so hold the condom firmly at the base of the penis so that it does not slip off, to ensure that no sperm can enter your partner's vagina or anus.
7. Do not throw the condom into the toilet. Ensure that the condom is safely disposed of and cannot pose a risk of infection to others.
8. When having sex again, always use a new condom for reasons of hygiene and safety.
9. If a condom should become physically damaged during use, seek medical advice within 48 hours.

Important information

- Store condom in a cool, dry place and protect against sunlight.
- Do not reuse. Re-use of condoms increases the risk of condom breakage and slippage.
- Condoms are produced from Natural rubber latex (except ceylor Non-Latex Ultra Thin), which can cause an allergic reaction in some cases. Allergy sufferers should preferably use ceylor Non-Latex Ultra Thin.
- Only use condom up to the date stipulated on the packaging.

- Do not carry out self-tests as this may damage the condom. ceylor condoms are tested according to leading-edge methods.
- If, during sex, you detect that the condom has shifted too high and may slip off, or it is too tight and may tear, stop and check the condom. A well-seated condom covers the entire erect penis, but is not overly creased.
- If the condom pod is visibly damaged then throw the condom away and use a new condom from an undamaged pod.
- Only use lubricant gels that are labelled suitable for use with condoms. Lubricants containing grease, such as Vaseline, baby oil, body lotions, massage oils, butter, margarine, etc. must not be used because they impair the characteristics of the condom.
- If over-the-counter or prescription medications are used on the penis or vagina, which could come into contact with the condom, ask a doctor or chemist about compatibility.
- If used correctly, ceylor condoms offer effective protection against undesired pregnancy and sexually transmitted diseases. However, no condom can guarantee 100% safety.

Blue Band – the classic one.
Experience passion thanks to the long-term classic for a sensual love life.

Easy Glide – the gliding one.
Experience an enhanced sensation thanks to the especially easy gliding condom for an even more intense, natural feeling.

Strawberry – the flavoursome one.
Experience added fun during sex thanks to the condom with strawberry flavour for a thrilling release.

Extra Feeling – the pearled one.
Experience an even greater thrill thanks to the pearled condom for intensive lovemaking.

Large – the big one.
Experience greater size thanks to the condom with an extra wide opening for maximum loving.

Extra Strong – the extra strong one.
Experience greater endurance during sex thanks to the extra strong condom for the powerful lover, for drawn out lovemaking.

Tight Feeling – the V-shaped one.
Experience greater hold thanks to the firmly seated condom with a tighter opening.

Fun Pack – for diversity.
Experience greater diversity thanks to the Fun Pack with a selection of condoms for richly varied lovemaking.

Thin Sensation – the real feel.
Experience intensive sensations and intense closeness thanks to the extremely delicate, extra sensitive condom for natural, authentic lovemaking.

Experience the bigger size and sensation of a condom with an extra wide opening AND 30% more lubricant for a more intense, natural feel.

Experience more happy moments thanks to Rain-bow Love with different condoms for varied lovemaking.

Non-Latex Ultra Thin – the ultra thin one.*
Experience more closeness thanks to the ultra thin non-latex condom for a vivid sensation during lovemaking. Polyurethane condoms.

CE 0123
 www.ceylor.com

Importeur/Importateur/Importatore:
Doethers Grether AG
Sternengasse 17
CH-4051 Basel
Switzerland

Effectum CH-REP AG, Kirchgasse 11, CH-4400 Olten, Switzerland

Sagami Europe, «Château de Brugheas», 12c Rue du Château, 03700 Brugheas, France

Sagami Rubber Industries Co., Ltd., 2-1, Moto-cho, Atsugi-shi, Kanagawa-ken, 243-0002, Japan

*** Non-Latex Ultra Thin:**
Sagami Manufacturers Sdn. Bhd., 31000 Batu Gajah, Perak/Malaysia

Rev.-Nr.: 06/2022/03/BPZ ceylor ES6344/4002