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
Amor Gummiwaren GmbH
August-Rost-Str. 4,
99310 Arnstadt, Germany

www.amor.ag
info@amor.ag
www.amor-shop.de
www.vibratissimo.com
www.RimbacherLatex.de

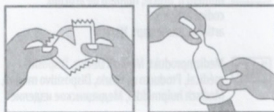CE 0197

DE Wie man(n)'s macht ...

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UK Instructions for use

How to use...

Check the integrity of the individual condom pack (foil wrap). If it is obviously damaged or open, discard the condom and use a new condom from an undamaged pack. Put the condom on before the first contact between the penis and the partner's body. To this end, carefully remove the condom from the foil with your fingers in order to prevent damage from finger nails, jewellery, etc. Squeeze the reservoir (the tip of the condom) with your thumb and index finger so that it no longer contains any air. Retract the foreskin completely, place the condom on the erect penis with the rolled edge to the outside and roll it down the entire penis. One should stop and check if one feels the condom sliding down or that it is excessively tight on the penis, as this can lead to a tear in the condom. To this end, pull the penis out. In the process, hold the condom on the root of the penis. Pull the penis out in the same way soon after ejaculation, but, at the latest, before it begins to slacken. Use the condom only once. If used again, there may be an increased risk of failure of the condom or of an infection. Dispose of used condoms in the household waste and not in the toilet.

What else you should know...

These condoms are made of natural rubber latex, which may induce allergic reactions including anaphylactic shock in the case of persons who are allergic to latex. Use as additional lubricants only such that are identified as suitable for use with condoms. Do not use greasy lubricants, such as vaseline, baby oil, body lotions, massage oil, butter, margarine, etc., as these damage the condom. Never use together with oily or greasy lubricants or vaginal suppositories. In the event of simultaneous use of medication that may come into contact with the condom, a doctor or pharmacist should be asked regarding its compatibility with condoms. Medical assistance is to be sought as quickly as possible, but at least within 72 hours, should a condom become leaky or crack during use or should incompatibilities arise.

Some further info...

Store condoms in their original packaging, in a cool, dry place and protected from sunlight. Please heed the batch number behind the batch code symbol and the expiry date behind the use by symbol on the packaging. After the expiry date, the condoms can no longer be used. Condoms are medical devices whose purpose is to provide a supporting effect in the prevention of sexually transmitted infections and pregnancy. No type of contraceptive guarantees 100% protection against pregnancy or infections.

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