body lotions, baby oil, butter, margarine, petroleum jelly etc. as anything containing oil can damage Latex condoms. Only use lubricants compatible with condoms. Some medications applied can damage condoms, so always check with your doctor or pharmacist.

7. After sex, and while the penis is still erect, carefully withdraw.

Hold the rim of the condom at the base of the penis to prevent any spillage of semen. Point the penis downwards and carefully slip the condom off.

8. Tie off the end of the used condom and wrap it in a tissue. Dispose of it hygienically. Do not flush the condom down the toilet.

⊗ CONDOMS SHOULD ONLY BE USED ONCE. Risks involved in re-using condoms include cross infection, tearing and loss of lubrication and strength through cleaning, all increasing the risks of pregnancy and STIs.

⊗ CAUTION: This product is made from natural rubber latex which may cause allergic reactions including anaphylactic shock.

Proper use of condoms will help reduce the risk of pregnancy, STIs and HIV/AIDS. Condoms are an efficient barrier to prevent pregnancy and STI's but, as with all contraceptives, 100% protection cannot be guaranteed. Non-vaginal use of condoms may increase the risk of pregnancy and STIs.

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⊗ MEDICAL DEVICE: Any serious incident that occurred in relation to this medical device may be reported to the manufacturer and to the Competent Authority of your country of residence.

⊗ Manufacturer legal entity

⊗ Authorised Representative in the European Community

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1. Open the wrapper, removing the condom carefully. Condoms are strong but they can be damaged by sharp finger nails, teeth or jewelry.

2. Always put the condom on the penis before any sexual contact occurs and only when the penis is erect. This will reduce the risk of unintended pregnancy and Sexually Transmissible Infections (STIs).

3. The condom will only unroll one way. Do not fully unroll the condom before use. Check that the condom is the right way up.

4. Gently pinch the tip of the condom to push out trapped air and to make room for semen. Any excess air trapped in the condom may increase the risk of condom breakage.

5. Roll the condom down the length of the penis to its base. If you feel the condom slipping or stretching stop and check the condom to avoid potential breakage.

6. Do not use oil based lubricants such as massage gels,

body lotions, baby oil, butter, margarine, petroleum jelly etc. as anything containing oil can damage Latex condoms. Only use lubricants compatible with condoms. Some medications applied can damage condoms, so always check with your doctor or pharmacist.

7. After sex, and while the penis is still erect, carefully withdraw.

Hold the rim of the condom at the base of the penis to prevent any spillage of semen. Point the penis downwards and carefully slip the condom off.

8. Tie off the end of the used condom and wrap it in a tissue. Dispose of it hygienically. Do not flush the condom down the toilet.

⊗ CONDOMS SHOULD ONLY BE USED ONCE. Risks involved in re-using condoms include cross infection, tearing and loss of lubrication and strength through cleaning, all increasing the risks of pregnancy and STIs.

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