JUST SAFE STANDARD

Standard condoms with an anatomic shape fit perfectly. These include lubricant and a reservoir.
Length 190 mm Nominal width 56 mm

FEEL SAFE ULTRA THIN

Ultra-thin condoms for even more feeling. Just as safe as normal condoms.
Length 180 mm Nominal width 54 mm

INTENSE SAFE RIBS & NOBS

Condoms with ridges and bumps for maximum stimulation. Contour shape.
Length 190 mm Nominal width 52 mm

SUPER LUBE EXTRA LUBRICANT

Standard condoms with an anatomical shape and reservoir. Provided with extra lubricant.
Length 190 mm Nominal width 56 mm

ALSO AVAILABLE

LUBRICANTS MASSAGE PRODUCTS & TOYS

www.safe.nl

Follow us #enjoysafe

MEDICAL DEVICE PRODUCT

CE
0123

safe
CONDOMS

INSTRUCTIONS FOR USE GEBRUIKSAANWIJZING / MODE D'EMPLOI GEBRAUCHSANWEISUNG / MODO DE EMPLEO

Manufactured by:
Thai Nippon Rubber Industry Public Company Limited
1 Charoenrat Road | Thungwatdon | Sathon |
Bangkok 10120 | Thailand

A Brand of Concepts of Love | Tomatenmarkt 1 |
1681 PH Zwaagdijk | The Netherlands (EU) |
+31 (0)228 82 00 00 | info@enjoysafe.com

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EN ISO 4074:2015

In the event of any serious incident has occurred in relation to the device please you should report it to the manufacturer and the competent authority or EC REP.

KING SIZE XL EXTRA LONG & WIDE

Extra long and wide condoms for those who need them.
Length 200 mm Nominal width 60 mm

PERFORM SAFE PERFORMANCE

Performance condoms to delay the climax on a natural basis.
Length 190 mm Nominal width 56 mm

SUPER STRONG FOR EXTRA SAFETY

Strong condoms for extra safety. Suitable for anal sex.
Length 190 mm Nominal width 56 mm

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ENGLISH

Use SAFE condoms This package leaflet includes important information! It is worth taking your time to read it carefully!

Condom safety Subject to correct use, condoms offer good protection from pregnancy and most STDs. However, no method can offer a 100% guarantee. In the EU, condoms can only be sold if these carry a CE mark. This means these condoms comply with strict requirements. The use of condoms is at one's own risk. Safe BV cannot be held liable for any damage.

Pay attention Always keep condoms in the original packaging, at a cool and dry place and out of reach of sunlight. No contraceptive offers 100% protection from capital letter or pregnancy. Oilbased lubricants or massage oil can damage condoms. Therefore, only use lubricants that are appropriate!

Instructions for use Make sure the use by date has not expired. Open the packaging and take out the condom. Be careful with sharp nails and/or objects! Squeeze the tip of the condom to remove any air. Check to which side the condom rolls down and then roll the condom over the stiff penis before any contact occurs between the penis and the partner's body to assist in the prevention of the sexually transmitted infections and pregnancy. After ejaculation, pull the penis back before it gets soft. While doing so, hold the rim of the condom to prevent it from slipping off. Dispose of the condom. Never flush a condom down the toilet. The condom is for single use and that if reused there might be an increased risk of failure or infection.

- Stop and check if the use feels the condom slipping off or tightening excessively onto the penis because this might lead to breakage.
- Consult a doctor or pharmacist about the compatibility of topical medicines that may come in contact with the condom.
- Seek medical assistance as soon as possible, at least within 72 h, should a condom leak or burst during use.
- Advice that if the individual container is obviously damaged to discard that condom and use a new one from an undamaged package.

Caution: This product is made of natural rubber latex, which might cause allergic reactions, including anaphylactic shock, if the user is allergic to latex.

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