prevenire gravidanza o malattie sessualmente trasmesse (STD).

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MD

260123_REV1

① Use the condom with the most caution. Delicately tear the wrapping on the serration side. Never use any cutting utensils and avoid sharp nails, teeth, and jewels that may damage the condom. Press the foil in order to get the condom out easily.

② Check the proper unrolling direction of the condom before wearing it.
③ Be sure to press the test end between the thumb and index finger to remove the air inside the condom so it can receive the sperm. Any excess air inside the condom may increase the risk of rupture. While pressing the end test between your fingers, place the condom on the tip of the erect penis and unroll it with another hand.

④ Unroll the condom all the way down the penis.

Instruction for use

1. Wear the condom on the erect penis before any contact between the penis and the partner's body to prevent pregnancy or sexually transmitted diseases (STDs).
2. If the condom is not worn correctly (if the air is not totally removed) if it is outside out if it comes off during intercourse, do not try to wear it on again, it may already have sperm on it; throw it away and use a new one. If the condom begins to roll on itself put it back immediately in place by dropping it again before continuing the sexual relationship.
3. When using a condom for anal intercourse, it is recommended to add water-based lubricant gel to prevent the condom from slipping or tearing. If you initially use a condom for oral use, make sure you use a new one for any further sexual intercourse.

These condoms are lubricated with a non-spermicidal silicone-based lubricant. Never use oil-based lubricants like Vaseline, baby skin oil, body creams, massage oils, cooling oil, butter, margarine, etc.

4. Dispose of the condom hygienically; wrap it up in tissue and throw it away in the bin. Never flush into toilets.
5. Remove the condom completely after ejaculation and before the end of the erection by firmly holding the condom in place at the base of the penis to avoid losing it or letting sperm escape. Remove the condom, take care to avoid contact with the penis, and used the condom with the body of the partner.

Precaution for use

1. These condoms are made of natural rubber latex, which can cause allergic reactions, including anaphylactic shock. If you are allergic to natural latex, do not use the condom until you have consulted a doctor or a pharmacist. Stop the use of the condom if you or your partner have any feelings of discomfort or irritation. If symptoms persist, consult a doctor.
2. When using prescription medicine applied on the penis, vagina, or anus, always check compatibility with the condoms by asking your medical doctor or your pharmacist.
3. A condom is used only once. Use a new condom every time you have sex. Reuse increases the risk of

lubrication and integrity of the condom due to cleaning increasing the risk of pregnancy and contamination by sexually transmitted diseases (STD).

4. Condoms should be always stored in a cool, dry place, away from direct sunlight. Never leave them in places exposed to hot weather or light (car, wallet, purse). The heat can deteriorate the physical properties of the condoms.
5. If the date has passed or if the packaging is damaged, do not use condoms.
6. If you think there is a risk that you or your partner may be pregnant or infected, consult your doctor immediately, at most within 72 hours after the incident.

Condoms reduce the risk of sexually transmitted diseases (STDs). Unfortunately, no condoms can guarantee 100% contraception and protection against sexually transmitted diseases (STDs).

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