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**ENG** Take your time and read carefully how to properly use a condom in order to enjoy a pleasant experience without the constraint of unpleasant surprises. Condoms are the only contraceptive method that when used properly reduce both—unwanted pregnancies and sexually transmitted diseases (STDs). Just bear in mind the following advice. Remember to always wear a condom before intercourse. Put on a condom upon erection and do not wait to do so right before ejaculation takes place, for seminal fluids may be secreted during intercourse. Use only water-based or silicone-based lubricants along with condoms. DO NOT use oil-based lubricants like vaseline, or body lotions because there might be a risk of the latex getting damaged as a result. Condoms are made of natural rubber latex which may cause allergic reactions including anaphylactic shock if the user allergic to latex.

#### BEFORE USAGE

1. Keep condoms in a cool and dry place. Avoid exposing them to sunlight, or putting them away in wallets or pockets where they may be subject to heat and/or friction.
2. Always check the expiration date before using a condom.

**ENG** Spend some minutes to read some indications on how to use a condom, it could give you a better sex and without worries, besides saving you from unpleasant surprises. The condom, if used correctly, is the only contraceptive method that at the same time reduces both the risk of contracting sexually transmitted diseases and unwanted pregnancies. It is enough to take a few small precautions. Remember to use the condom before any sexual contact, once you have reached erection, without waiting for the last moments of ejaculation, as even during preliminaries there can be premature discharge of seminal fluid. With the condom use only water-based or silicone lubricants. Do not use oil-based lubricants such as vaseline, creams and lotions for the body: they could damage the latex. Condoms are made of natural rubber latex which may cause allergic reactions including anaphylactic shock if the user is allergic to latex.

3. Remember to wash your hands before opening the condom.
4. Do not unfold the condom before putting it on, do so only after it is placed on top of the erect penis.

Lovematch condoms are dermatologically tested. Nevertheless, it is recommended to see a doctor in case of irritation or soreness. Condoms are made to be used only once. The non-vaginal use of condoms may result in having the condom slip off of the erect penis or having it break. In these cases, it is recommended to use a more durable condom or/and a lubricant.

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**ENG** Before opening the foil condom wrapper, remember to wash your hands. Open the condom wrapper starting at the serrated edge and carefully remove the condom. The condom must be handled with care: scissors, fingernails, or other sharp objects may damage and compromise its integrity. Extract the condom from the foil packaging and make sure that it is opened the right way. The latex must be facing outward (i.e. do not unroll the condom inside out).

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**ENG** Place the condom on the erect penis before coming in contact with your partner. If you realize that the condom is inside out, discard it and use a new one.

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**ENG** Place the unrolled condom onto the tip of the erect penis. At this point pinch the remaining tip of the condom using the thumb and index finger in order to prevent air from entering it. At the same time, while using the other hand, roll the condom down to the base of the penis. Check to see that the condom perfectly adjusts to the penis without any wrinkles or air bubbles, especially on the glans.

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**ENG** Immediately after ejaculating, remove the still-erect penis while firmly holding the condom at its base with your fingers: in this way you can avoid the condom slipping off the penis, or remaining inside the vagina. Throw the used condom away, and do not flush it down the toilet, as it is not biodegradable.

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rrt Vertrieb und Service GmbH,