Instructions for use for Latex Probe Covers

GB Pasante Latex Ultrasound non-lubricated probe covers are designed for easy application and fit various ultrasound transducer shapes. Transducer covers are a medical device used to prevent cross-contamination of transducer probe during endocavity (i.e. transvaginal or transrectal) ultrasound procedure. Intended for barrier protection during invasive medical examination and diagnosis using transducer probe. It is intended to be used by a healthcare provider trained in the use of ultrasound. There are no unique skills or user abilities required for protective transducer use. Prevent cross-contamination of probes and consequently reduce the risk of cross infection in patients undergoing invasive ultrasound examinations and diagnoses (i.e. transvaginal and transrectal scan). User group may include, but not limited to: Anesthesiologists, Department Nursing Staff, Emergency Medicine physicians, Endocrinologists, Radiologist, Sonographers, Surgeons, Surgical Technicians and other clinicians. Before use, you should be trained in ultrasonography. Disinfection of the transducer probe after examination could reduce the probability of cross-contamination. Store in cool, dry place away from direct sunlight. Report any serious incident occurred from using the device to the manufacturer and the relevant national competent authority. **DISPOSE** of single use components as infectious waste. **DO NOT REUSE** probe covers. Reuse or reprocessing may create a risk of contamination of the device and cause patient infection or cross-infection. **USE WATER BASED LUBRICANT**, avoid petroleum or mineral oil based lubricant that may harm the covers. This product contain natural rubber latex which may cause allergic reactions.

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How to use a Probe Cover

1. Carefully tear off the foil pack. Make sure not to damage the cover. **2.** During removal from foil, avoid damage to the cover from sharp edges, fingernails etc. **3.** Place the cover over the transducer. Pull cover tightly over the transducer to remove wrinkles and air bubbles, taking care to avoid puncturing the cover. Inspect the cover to ensure there are no holes or tears. **4.** After use make sure you dispose of the Probe Cover hygienically. Wrap it in tissue and place it in a waste bin.

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EU
Advena Services Ltd

EC REP
Advena Ltd

Tower Business Centre, 2nd Flr.,
Tower Street, Swatara, BKR 4013 Malta.

UK
Pasante Healthcare Ltd., Commerce Way,
Lancing, W Sussex, BN15 8TA, UK