CE
0123

EN ISO 4074:2015

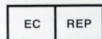
muchach

Distribuito da: / Distributed by: / Distribuido por: / Distribué par: / Vertriebs:

ITA S.r.l.
Via G. Bortolan, 28
31050 Vascon di Carbonera (TV) Italy
Tel. +39 0422 4476 - Fax +39 0422 447777
www.itagency.it - ita@itagency.it - export@itagency.it

MADE IN PRC

Shanghai International Holding
Corp. GmbH (Europe)
Eiffestraße 80
20537 Hamburg
GERMANY

Fabbricante: / Manufacturer:
Fabricante: / Hersteller: / Fabricant:
DongYang Songpu Latex(Jinzhou) Co., Ltd.
New and High-tech development Zone
121000 Jinzhou, Liaoning Province
PEOPLE'S REPUBLIC OF CHINA

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EN - READ THE FOLLOWING INSTRUCTIONS CAREFULLY BEFORE USE

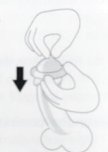
Place the condom over a fully erect penis before sexual intercourse.

Open the aluminum package carefully. Do not use sharp objects and take care not to damage the condom with fingernails or anything else.

Place the rolled condom over the tip of the penis and pinch the air out of the reservoir top with one hand. Using the other hand gently unroll the condom down over the whole penis.

Immediately after ejaculation, pull out before the penis softens while holding the condom against the base of the penis to make sure that the semen does not spill.

Remove the condom and dispose of it in the waste bin. Do not flush it down the toilet. Condoms must never be reused. A new condom must be used every time there is sexual intercourse.

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CAUTION

Persons allergic to latex must not use this condom.

It must be unrolled over the penis; a condom that has already been unrolled must not be used.

It must be used immediately after the sealed package has been opened.

It must be kept sealed inside the package.

It must be disposed of after use, but not down the toilet since it is not biodegradable.

Do not use the condom if it is dry, sticky or clearly deteriorated / damaged.

Store the condom in a cool, dry place out of direct sunlight.

If used correctly a latex condom can prevent pregnancy and protect against many sexually transmitted diseases such as syphilis, gonorrhea, chlamydia, genital herpes, mycosis, hepatitis B and AIDS. Using a condom does not, however, fully eliminate the risk.

In the event of breakage, if the purpose of the condom was to prevent pregnancy, seek medical advice within 72 hours.

Do not use oil-based lubricants, which could weaken the latex. If extra lubrication is required, use water-based lubricants.

In rare cases or in hypersensitive persons, the condom may cause ACD (allergic contact dermatitis) to male and female genitals and the skin and mucosal surfaces around them.

In such cases it is advisable to seek medical advice.

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