ES

Como se usa ...

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EN

How to use ...

Put condom on to the erect penis before there is any contact between the penis and your partner's body. Carefully remove the condom from the foil with your fingers in order to avoid damage from fingernails, jewellery etc.. Pinch the reservoir (tip of the condom) between thumb and index finger to squeeze out the air. Pull back the foreskin completely, put on the condom and roll it down over the whole length of the penis. Withdraw the penis while it is still erect, holding the condom in place at the base of the penis. Only use the condom once. In case of re-use, contrary to the specifications, the assigned purpose of the condom (contraception and protection against sexually transmitted diseases) can no longer be assured.

What else you should know ...

These condoms are made of natural rubber latex. In cases of known oversensitivity to latex (allergy), even natural rubber latex products are not recommended. Only use additional lubricants that are identified as being suitable for use together with condoms. Avoid oil-based lubricants such as Vaseline, baby oil, body lotions, massage oil, butter, margarine etc., as they will damage the condom. In the case of simultaneous use of medicines, which could come into contact with the condom, please consult a medical practitioner or pharmacist regarding condom compatibility. Consult a medical practitioner even in cases of slight irritations, e.g. skin irritations. Used condoms must be disposed of appropriately (into the waste bin) and not flushed down the toilet.

Some further info ...

Store the condoms in their original packaging in a cool, dry place well away from sunlight. Please be aware of the batch number (next to LOT- symbol) and the expiry date (next to egg timer-/EXP-symbol) on the package. The condoms may no longer be used after the expiry date. Condoms are Medical Devices for the purpose of contraception and to support the protection against sexually transmitted diseases. No type of contraception guarantees 100% protection against pregnancy or infections.

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