**POWERS TO KNOW BEFORE USING A CONDOM 1.** If extra lubrication is needed, use water-based lubricants. 2. DO NOT USE OIL BASED LUBRICANTS such as petroleum jelly, mineral oil and cold cream, as they may weaken the condom and cause subsequent failure of the product. So can some topical medicines that are applied to the penis or vagina. If you are not sure about the effect of the medicines you using on condoms, ask your doctor or pharmacist. 3. Do not use the condom if the rubber material is sticky or brittle or the individual container is obviously damaged. 4. Seek medical assistance as soon as possible, at least 72 hours after a condom leak or break. 5. Condoms are for single use only, reuse cannot help increase the risk of leakage and infection. 6. Use when correctly every time you have sex, latex condoms help prevent pregnancy, reduce the risk of transmitting HIV/AIDS and other sexually transmitted infections. 7. No method of contraception can provide 100% protection against pregnancy or transmission of HIV and STIs. If you are pregnant or you or your partner may have become pregnant, you should use another method of contraception. 8. If you are allergic to latex, you should use condoms that cause allergic reactions including anaphylactic shock. If you are allergic to natural rubber latex products, consult your doctor for advice before use. 9. Store in a cool dry place away from direct sunlight. 10. Keep away from children. Can choke or suffocate on uninflated or broken condom fragments. 11. Use of condom during oral sex increase the risk of asphyxiation or choking. 12. The use of additional lubricants during and/or sex can reduce the risk of leakage. 13. Condom is a medical device to be used by persons - adult male or female. 14. For more information, visit our website [www.moh.gov.sg](http://www.moh.gov.sg) or contact our helpline at 1800 471 2222. Manufacturer and the competent authority of the Member State in which the user and/or patient is established. 15. These condoms conform to International standards ISO 4074/EN ISO 4074.

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