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EN **How to use Glyde condoms** **Use condoms only once - do not reuse!**

- Open the condom foil carefully. Take care not to damage the condom with fingernails, rings or body jewelry.
- Do not unroll the condom before putting it on. Unrolling the condom before use may damage it.
- Put the condom on before any contact occurs between the penis and the partner's body.
- Gently hold the reservoir tip of the condom between thumb and forefinger (this makes sure there is no air trapped in it) and unroll the condom onto the erect penis, rolling smoothly all the way down.
- After you have put on the condom, add plenty of lubricant if desired; take care only to use water-based, condom-friendly lubricants because others might damage the condom.
- Withdraw the penis from your partner soon after ejaculation, before the erection goes down. Hold the rim of the condom round the penis so that the condom does not slip off during withdrawal; this should prevent spillage.
- After withdrawal, take care not to touch the vaginal area with either the penis or the condom.
- If the condom breaks or slips off during intercourse don't ejaculate inside your partner, withdraw immediately.
- Check with a doctor or a pharmacist about the compatibility of condoms and topical medicines applied to the penis, vagina or rectum.
- Storage:** Store condoms below 25 degrees C in a cool dry place away from direct sunlight. Do not use the condom after the use-by date indicated on the foil or package. Dispose of used condoms by wrapping and putting in a garbage bin. Do not flush down toilet.
- Side-Effects and Incompatibility:** Condoms are a mechanical barrier and do not interfere with your organism. However, condoms made from natural rubber latex should not be used by people having an allergy or cross-allergy against latex or whose skin reacts or aches when getting in touch with latex. If in doubt, consult your physician as soon as possible and use non-latex condoms.
- Prevention of Pregnancy and STI Transmission:** Although no contraceptive can guarantee 100% effectiveness, condoms when properly used can help prevent pregnancy and reduce the risk of AIDS and the spread of many sexually transmitted infections (STIs), but cannot totally eliminate the risk of acquiring these diseases. STIs as the name suggests are usually contracted as a result of sexual contact. Any person evidencing symptoms should consult their doctor, STI/GUM clinic or hospital as soon as possible.

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