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en Each condom should only be used once.
Reusing a condom reduces its safety and
could lead to unwanted pregnancy and/or
infection with HIV or contagious sexually
transmitted diseases.

Please check the expiry date on the package
and the condom wrappers – do not use the
condom if expired! Store condoms in a cool
and dry place protected from heat and direct
sunlight. Condoms do not interfere with the
human organism and have no side effects.
However, intolerance may occur in the form
of allergic reactions (e.g. latex allergy).
If you experience any discomfort or irritation, please stop
using the condom and seek medical advice.
MAPA condoms are exclusively intended for
vaginal intercourse. If used correctly, MAPA
condoms provide protection from pregnancy
and transmission of diseases. However, no
contraceptives completely eliminate the risks
of pregnancy or infection. Do not use with
oil-based or greasy lubricants. If lubrication is
required, use lubricants that are approved for
use with condoms. Before using a lubricant,
please read the instructions on the packaging.
Do not apply lubricant directly to the penis or
the inside of the condom, because this increases
the risk of the condom slipping or
tearing. If used with vaginal suppositories,
please observe the respective instructions
for use. If prescription or non-prescription
medication (e.g. ointments, gels etc.) is applied
to either the penis or vagina, please check with
your doctor or pharmacist for compatibility
of the medication with the condom. Caution:

Condoms can be damaged by jewellery or
body piercings. If the condom has torn or
broken during use, seek medical advice as
soon as possible, but within 72 hours.

Please report any serious incident in relation
to the device to the manufacturer and
your national competent authority.

1 Carefully tear open the condom
wrapper at the marking "open here".
The wrapper can thus be torn open cleanly
without damaging the condom. Caution:
The condom may be damaged by sharp
objects, jewellery or fingernails. Do not use
the condom if the wrapper is damaged or
already open.

2 Hold the condom taut between the
thumb and fore-finger, making sure
that no air is trapped inside. Pull back the
foreskin and unroll the condom all the way
down the erect penis before any contact of
the penis with the body of your partner.
Only then can the condom help prevent
sexually transmitted infections and
pregnancy. If the condom does not unroll
easily, it may be on backwards. In this case,
do not turn it around for use as your partner
may come into contact with body
secretions. Throw it away and use a new
condom, making sure that it is on the right
way for unrolling. If you notice that the
condom slips or tightens, check it for
potential damage as this may cause a tear.

3 Before the penis becomes flaccid, pull
the penis out of the vagina. To do so,
hold the condom against the base of the
penis as you gently pull the penis out of the
vagina. This prevents the condom slipping
off.

de Used condoms should be disposed of
in a waste bin, not down the toilet.
Wash penis and hands before resuming
sexual contact. Condoms are designed to
be used once only.

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MAPA GmbH, Industriestraße 21–25