PRECAUTIONS:
If there are lesions, open sores, or viral warts noticeable on either partner, DO NOT ENGAGE IN SEXUAL PENETRATION AND CONSULT YOUR DOCTOR.

WARNING:
FOR MAXIMUM PROTECTION IT IS IMPORTANT TO FOLLOW THESE INSTRUCTIONS CAREFULLY.

INSTRUCTIONS FOR CONDOM USE

Manufactured to EN ISO 4074:2015

LTC Healthcare
LTC Healthcare Customer Care
Tel No: 00 44 (0) 1423 326687
European Distributor: LTC Healthcare, The Old Station Yard, Milby Road, Boroughbridge, YO51 9BL
T: 00 44 (0) 1423 326687 E: sales@ltchealthcare.co.uk
www.ltchealthcare.co.uk
@exscondomofficial @exscondoms

Thai Nippon Rubber Industry Public Company Limited
1 Charoenrat Road, Thungwatdon, Sathon, Bangkok, 10120, Thailand
MDSS GmbH
Schiffgraben 41, 30175 Hannover, Germany
MD

UK

INSTRUCTIONS FOR CONDOM USE

1. Check the date on the condom foil to ensure the condom is within the expiry date usually printed on the back of the foil. Tear along one side of the condom foil making sure you do not damage the condom with finger nails or jewellery. If the condom foil is damaged discard that condom and use a new one from an undamaged pack.
2. If you are using a lubricant only use a water based lubricant such as EXS™ or Boys Own™ do not use oil based lubricants such as petroleum jelly as these may damage the condom.
3. Hold the condom feat between your finger and thumb and roll the condom onto an erect penis before contact with your partner, to assist in the prevention of sexually transmitted infections and pregnancy. Unroll the condom gently down the full length of the penis expelling any air from the condom.
4. If you feel the condom is slipping off, or is tightening excessively then withdraw the penis from your partner and check the condom is in place and not damaged. If the condom has moved then roll it back in place or use another condom if you are concerned about the condom being damaged.
5. After ejaculation withdraw the penis from your partner holding the base of the condom to ensure it does not slip off. When the penis is withdrawn from your partner you can then remove the condom.
6. ⚠️ Condoms are made of natural rubber latex which may cause an allergic reaction including anaphylactic shock if the user is allergic to latex.
7. If you are concerned the condom has been damaged during sex then seek medical assistance such as a Doctor or Sexual Healthcare advisor preferably within 24 hours, but not later than 72 hours. If the condom foil is damaged discard that condom and choose another condom from the pack. Consult a doctor or pharmacist about the compatibility of topical medicines that might come in contact with the condom.
8. Use a condom only once. Re-use of a condom may increase the risk of breakage, slippage, transmission of sexually transmitted infections including HIV. Dispose of the condom by wrapping it in tissue and placing in a waste bin. Store any unused condoms in a cool dry place away from direct sunlight.
9. Latex condom are intended to prevent pregnancy, HIV/AIDS and other sexually transmitted infections.
10. In the event of any serious incident has occurred in relation to the device, please you should reported it to the manufacturer and the competent authority or EC REP.

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